

MECTRON OZOACTIVE

ASSEMBLY PIEZOSURGERY® / OZOACTIVE



OzoActive produces ozone and injects it directly into PIEZOSURGERY® device irrigation lines, providing ozonated water which reduces the risk of fungi, bacteria and virus contamination through aerosols.



1 OZOACTIVE

Connect foot-pedal via the jack terminal



2 OZOACTIVE

Connect the power cord

mectron

medical technology

MECTRON OZOACTIVE



3 OZOACTIVE

Connect the thinner tube of the Oral Surgery irrigation kit into the designated coupler



4 PIEZOSURGERY®

Disconnect foot-pedal cable, pump tubing components from infusion tube and handpiece irrigation coupler



5 PIEZOSURGERY®

Connect foot-pedal via the Ethernet plug port



6 PIEZOSURGERY®

Connect irrigation tube labeled "HANDPIECE" to Handpiece irrigation coupler

NOTE: Arrow indicates the direction of water flow (*)




7 PIEZOSURGERY®

Insert irrigation tube into peristaltic pump




8 PIEZOSURGERY®

Connect tube to infusion line

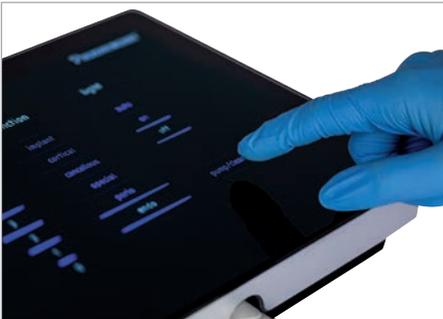


9 PIEZOSURGERY®

Prime the irrigation line by activating a Pump/Flush cycle

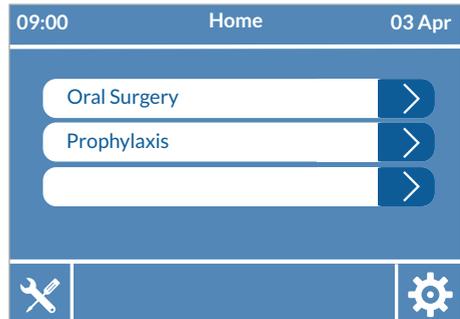

10 OZOACTIVE

Choose the Oral Surgery function

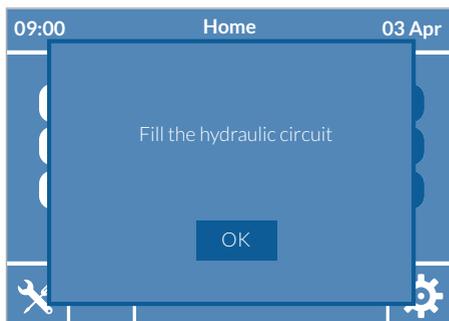

11 OZOACTIVE

Verify that the irrigation line has been fully primed and press OK



12 OZOACTIVE

Adjust the level of ozone according to the irrigation level on the device

IRRIGATION LEVEL	OZONE LEVEL
1	1
2	2
3 (default)	3 (default)
4	4
5	5
6	6



13 SETTINGS TABLE

Set the ozone level in accordance with the irrigation level of the PIEZOSURGERY® device. See indications table

14 PIEZOSURGERY® / OZOACTIVE

Press foot-pedal to start

⚠ WARNING

- PIEZOSURGERY® and OzoActive default setting level is 3.
- In the Oral Surgery function, ensure that the irrigation kit is connected according to the water flow indication arrow.
- Do not use the device without irrigation and without first priming the irrigation line.